

CD13 antibodies

Analyte specific reagents (ASR)

Analytical and performance characteristics were not established



Miltenyi Biotec B.V. & Co. KG

Friedrich-Ebert-Str. 68
51429 Bergisch Gladbach
Germany
☎ +49 2204 8306-8484
🌐 www.miltenyibiotec.com

1. General information

Intended use

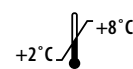



REA263 reacts with human CD13. The fluorescently labeled CD13 antigen can be detected by flow cytometry.

Reagents and contents

Monoclonal CD13 antibody conjugates

Product	Volume	REF
CD13-PE	1 mL	170-081-019

2. Technical data and background information

Antigen	CD13
Clone	REA263
Isotype	Recombinant human IgG1, κ light chain
Alternative names of antigen	gp150, Myeloid plasma membrane glycoprotein CD13, Microsomal aminopeptidase, Alanyl aminopeptidase, Aminopeptidase N
Purification	Affinity chromatography
Product formulation	Antibodies are supplied in buffer containing stabilizer and 0.05% sodium azide.
	Store at +2 °C to +8 °C (+36 °F to +46 °F).
	Store protected from light.
	The use-by date is indicated on the vial label.
	For in-use stability at +2 °C to +8 °C (+36 °F to +46 °F) storage temperature refer to the use-by date indicated on the vial label. Do not use the reagent after the use-by date.

Expression pattern

REA263 recognizes the human CD13 antigen, a 150–170 kDa type II transmembrane glycoprotein, which is also known as aminopeptidase N or gp150. CD13 is expressed on granulocytes, myeloid progenitors, endothelial cells, epithelial cells, and subset of granular lymphoid cells. It is also broadly expressed in other tissues such as kidney proximal tubules, intestine, and placenta. CD13 is an enzyme that is used as a biomarker to detect damage to the kidneys, and that may be used to help diagnose certain kidney disorders. It also serves as a receptor for one strain of human coronavirus that is an important cause of upper respiratory tract infections. Defects in CD13 appear to be a cause of various types of leukemia or lymphoma. Additional information: Clone REA263 displays negligible binding to Fc receptors.

3. Warnings and precautions

- ▲ Interpretation of results is under the full responsibility of the user.
- ▲ For all handling, consideration of good laboratory practice (GLP) regulations is recommended.
- ▲ Use of the reagents is restricted to trained and qualified personnel only.

- ▲ All biological specimens and all materials that come into contact with blood and blood products must be treated as infectious material. Regulations for the treatment and disposal of infectious material must be followed.
- ▲ Reagents contain sodium azide (NaN₃), a chemical highly toxic in pure form. However, at product concentrations, it is not classified as hazardous. Sodium azide may react with lead and copper plumbing to form highly explosive buildups of metal azides. Upon disposal, flush with large volumes of water to prevent metal azide build-up in plumbing. Safety guidelines must be observed.
- ▲ For material required but not provided the manufacturers recommendations and safety regulations must be followed.
- ▲ Reagents should not be used if signs of leakage are observed. Use undamaged and sealed vials only.

4. Application

Reagents can be used for immunophenotyping by flow cytometry and other research applications.

5. General Use considerations

Principle of method:

The antibody reagent provided enables the identification of a specific target cell type by flow cytometry. This technique is based on fluorochrome conjugated antibodies binding to specific antigens expressed by the target cells. Incubating a sample of interest, e.g., peripheral blood mononuclear cells (PBMC), with the provided antibody reagent leads to fluorescent staining of the cell type expressing the specific target antigen. Analysis of the sample is performed in a flow cytometer at a single-cell level. The analysis is based on the detection of characteristic light emission patterns emitted by the fluorescently labeled antibody upon excitation with laser light. The collected data can be processed and analyzed using flow cytometry software.

Important notes:

Exposure of reagents to temperatures below +2 °C (+36 °F) and above +8 °C (+46 °F) and to light should be minimized during handling.

Sample requirements

- Reagents can be used for determination of antigen-positive cells in whole blood samples by flow cytometry.
- Each cell source can have different storage conditions and limitations that should be considered prior to collection and analysis. For collection of patient samples national legislation must be followed.
- Whole blood samples should be stained within 24 hours.
- Viability of the cells should be assessed and use of samples with at least 80% viable cells is suggested in order to minimize risk of erroneous results.

Quality control:

It is recommended to run regularly a control sample from a normal adult specimen or commercially available whole blood control as a quality control of the system.

6. Analytical specificity

Analytical specificity was demonstrated by evaluating co-staining efficiency of the Miltenyi clone REA263 to a relevant reference reagent of the same specificity conjugated to a different fluorophore. In addition, clone REA263 was added as pure antibody to a staining performed with the reference reagent, the device under investigation and another antibody-fluorochrome of REA263 on 3 biological samples, respectively.

7. Excitation and emission data of fluorochrome conjugates

Fluorochrome	Excitation laser (nm)	Excitation maximum (nm)	Emission maximum (nm)
VioBlue®	405	400	452
VioGreen™	405	388	520
VioBright™ FITC	488	496	522
FITC	488	495	520
PE	488 or 561	565	578
PE-Vio®615	488 or 561	565	619
PerCP	488	482	675
PerCP-Vio®700	488	482	676
PE-Vio®770	488 or 561	565	775
APC	561 or 635	652	660
APC-Vio®770	561 or 635	652	775

8. Limitations

Combinations of recombinant human IgG, k light chain antibodies with anti-human IgG1 or anti-human Ig k light chain reagents can generate misleading results.











Use of monoclonal antibodies in patient treatment can interfere with recognition of target antigens by this reagent. This should be considered when analyzing samples from patients treated in this fashion. Miltenyi Biotec has not characterized the effect of the presence of therapeutic antibodies on the performance of this reagent.

Reagent data was collected typically with EDTA-treated blood. Reagent performance can be affected by the use of other anticoagulants.

9. References

1. Look, A. T. *et al.* (1989) Human myeloid plasma membrane glycoprotein CD13 (gp150) is identical to aminopeptidase N. *J. Clin. Invest.* 83(4): 1299-1307.
2. Favaloro, E. J. *et al.* (1993) CD13 (GP150; aminopeptidase-N): predominant functional activity in blood is localized to plasma and is not cell-surface associated. *Exp. Hematol.* 21(13): 1695-1701.
3. Yeager, C. L. *et al.* (1992) Human aminopeptidase N is a receptor for human Coronavirus 229E. *Nature* 357(6377): 420M22.

10. Glossary of symbols

	Manufacturer		Use-by date
	Order number		In-use stability
	Batch code		Temperature limit
	Consult instruction for use		Protect from sunlight
	Phone		Website

MACS, the Miltenyi Biotec logo, Vio, VioBlue, VioGreen and VioBright are registered trademarks or trademarks of Miltenyi Biotec B.V. & Co. KG and/or its affiliates in various countries worldwide.

Copyright © 2020 Miltenyi Biotec and/or its affiliates. All rights reserved.